





# Contour<sup>®</sup> plus ONE<sup>»</sup>

Blood Glucose  
Monitoring System

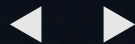


Using the  
**smartLIGHT** feature on the  
**CONTOUR®PLUS ONE**  
smart meter

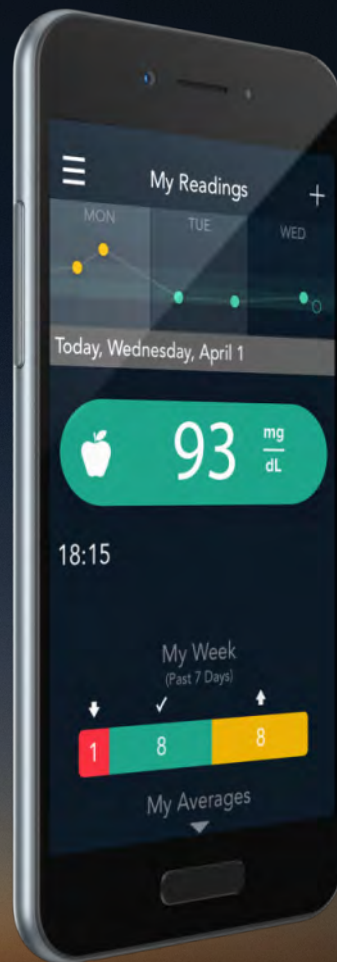

**Contour**  
**plus ONE**  
Blood Glucose  
Monitoring System




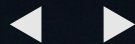

The **smartLIGHT** feature gives you instant feedback on your blood glucose results.



Contour<sup>®</sup>  
plus **ONE**»  
Blood Glucose  
Monitoring System

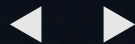

1.


## smartLIGHT Feature

After you have tested your blood glucose and if the Target Light setting is on, the test strip port light displays a color representing your result value compared to the Before Meal, After Meal, or Overall Target Range.



Contour<sup>®</sup>  
plus ONE<sup>»</sup>  
Blood Glucose  
Monitoring System



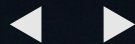


2.


## smartLIGHT Feature

If your blood glucose result is in target, the smartLIGHT feature is green and the meter beeps.

In Target

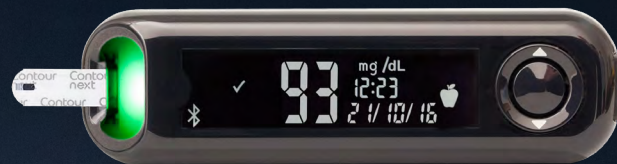

Contour<sup>®</sup>  
plus **ONE** »  
Blood Glucose  
Monitoring System

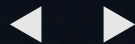


3.

### smartLIGHT Feature


If your blood glucose result is below target, the smartLIGHT feature is red and the meter beeps.

Below Target

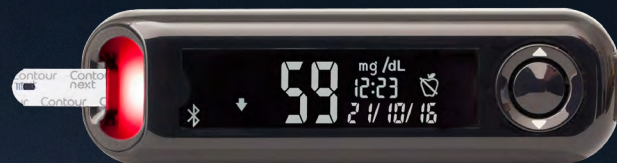

Contour<sup>®</sup>  
plus ONE<sup>»</sup>  
Blood Glucose  
Monitoring System



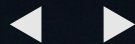

4.


### smartLIGHT Feature

If your blood glucose result is above target, the smartLIGHT feature is yellow and the meter beeps.

Above Target




Contour<sup>®</sup>  
plus ONE<sup>»</sup>  
Blood Glucose  
Monitoring System

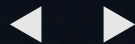

5.

## smartLIGHT Feature

If you have not selected a Meal Marker, the smartLIGHT feature will indicate your blood glucose test result compared to your Overall Target Range.

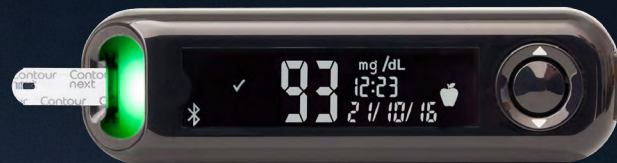

Contour<sup>®</sup>  
plus ONE<sup>»</sup>  
Blood Glucose  
Monitoring System



Talk to your Healthcare Professional and refer to the product user guide if you have questions about the **CONTOUR<sup>®</sup>PLUS ONE** smart meter and the **CONTOUR<sup>™</sup>DIABETES app**. For more information, and to find out whether the **CONTOUR<sup>®</sup>PLUS ONE** meter is available in your country, visit [diabetes.ascensia.com](http://diabetes.ascensia.com)





Get the app



Apple and the Apple logo are trademarks of Apple Inc., registered in the US and other countries. App Store is a service mark of Apple Inc. Google Play is a trademark of Google Inc.

Ascensia, the Ascensia Diabetes Care logo and CONTOUR are trademarks of Ascensia Diabetes Care Holdings AG.

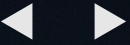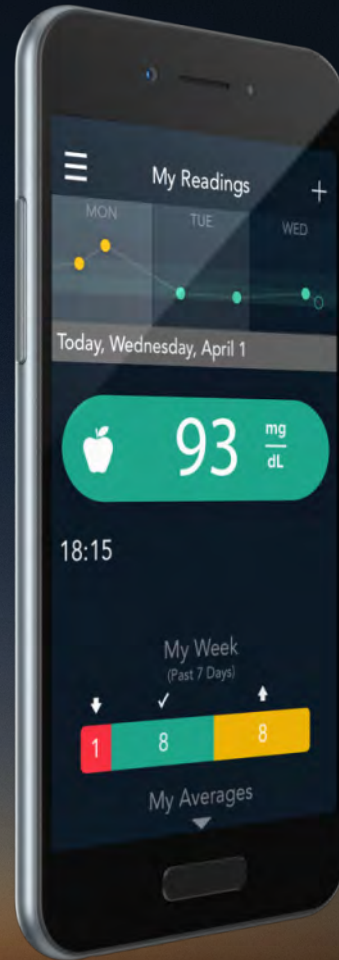

**Contour<sup>®</sup>  
plus ONE<sup>»</sup>**  
Blood Glucose  
Monitoring System



# Contour<sup>®</sup>

diabetes solutions